

Athelas is bringing White Blood Cell /Neutrophil% monitoring HOME

Athelas heard back from the FDA this past week and is in the home stretch to getting device clearance from the FDA. FDA clearance means that patients will get to monitor their blood count from the home. Less lab draws, more at-home monitoring. We are looking to partner with individuals who are immunocompromised, either as a result of an autoimmune disease (i.e, arthritis, lupus, multiple sclerosis, celiac,etc.) or chemotherapy. Patients who are in remission are also eligible.

Should you be willing to support, this is how the process would look like:

1. You will perform a finger prick test by yourself. You will receive training prior.
2. An operator will do another prick and run the same test.

2 finger pricks, 40 minutes.

Compensation: \$35 amazon gift card.

Testing will be done by a trained professional with requisite precautions for COVID 19. Please reply directly to **manuella@getathelas.com** if interested.

With so much appreciation and gratitude, Athelas.

Sample Collection



1) Disinfect fingertip.
Allow to dry before puncturing.



2) Twist the protective lancet tip
and pull off.



3) Lightly hold pressure at the
fingertip and puncture.



4) Wipe away the first drop with a
dry lint-free wipe.



5) Use the second drop to fill up the
slim column of the purple
micropipette.



6) Twirl micropipette for 20
seconds to even out blood
content.



7) Place the capillary on the inlet of
the test strip until the whole T
channel is filled.



8) Magnetic side first,
insert into the strip lot.